Ginex

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ENGLISH

The following instructions describe all GYNEX chairs versions and all possible accessories, and, because of this reason, not all paragraphs could be applied to your specific chair.

EUROCLINIC S.r.l. will supply to authorised assistance technicians all information that could be necessary for installation and repair.

It is prohibited to double, memorize and divulge this publication without written authorisation of EUROCLINIC S.r.l.

All kind of information, technical specifications and pictures, which you will find in the following pages, are not binding.

EUROCLINIC S.r.l. reserves itself the right to make modifications and make technical improvements without modifying the following instructions.

Original text of this manual is in italian and english language.

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IMPORTANT DIRECTIONS!

- PLEASE, DISCONNECT GENERAL SWITCH, BEFORE LEAVING OUTPATIENTS DEPARTMENT.
- THIS CHAIR IS NOT PROTECTED AGAINST FLUIDS PENETRATION.
- DO NOT USE THIS EQUIPMENT IN PRESENCE OF INFLAMMABLE ANAESTHETIC GAS MIXTURE WITH OXYGEN AND PROTO-OXIDE AZOTE.
- THE EQUIPMENT CAN BE ELECTRICALLY CONNECTED ONLY TO OTHER INSTRUMENTS PROVIDED WITH CE MARK.
- WITH REGARD TO PATIENTS WITH CARDIO-STIMULATOR OR HEARING PROTHESIS, PLEASE CONSIDER POSSIBLE EFFECTS OF USED INSTRUMENTS ON CARDIO-STIMULATOR ON HEARING PROTHESIS. IN THIS CONNECTION, PLEASE, KINDLY PAY ATTENTION TO TECHNICAL-SCIENTIFIC LITERATURE ABOUT THIS SUBJECT.
- THE EQUIPMENTS HAVE BEEN PROTECTED FOR CONTINUOUS WORKING WITH INTERMITTENT LOAD. YOU WILL FIND PRESCRIBED WORKING AND REST TIMES LISTED BELOW:
WORKING: 1 MINUTE
REST: 10 MINUTES
- EUROCLINIC S.R.L. GUARANTEES ABOUT SECURITY, RELIABILITY AND PERFORMANCES OF ITS EQUIPMENTS.
- WARRANTY IS CONDITIONED BY THE FOLLOWING INDICATIONS RESPECT: OBSERVANCE OF CONDITIONS WHICH YOU CAN FIND ON WARRANTY CERTIFICATE. THE EQUIPMENT MUST BE USED ONLY ACCORDING TO INSTRUCTIONS LISTED IN THIS MANUAL.
ELECTRICAL EQUIPMENT OF THE SURROUNDINGS WHERE THE CHAIR IS INSTALLED HAS TO BE UP TO STANDARDS C.E.I. (STANDARDS ABOUT ELECTRICAL EQUIPMENTS FOR MEDICAL PLACES). ASSEMBLY, REPAIRS, ENLARGEMENT OF EQUIPMENT, ADJUSTMENTS AND IN GENERAL ALL OPERATIONS ABOUT COFFER OPENING MUST BE MADE ONLY BY TECHNICIANS AUTHORISED BY EUROCLINIC S.R.L.

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DUE USE AND EMPLOYMENT WAYS

GYNEX chairs are medical equipments for gynaecological treatment.

Attention!

Maximum load of the chair is kg 135. This value must not be surpassed.

BIOCOMPATIBILITY

Stuffings are lined with Skay: you have to use only Euroclinic original spare parts.

SORROUNDINGS CONDITIONS.

The equipment has to be installed in surroundings with the following features:

- a) temperature between 10° and 40° C
- b) Relative dampness between 30 and 75%
- c) Atmosphere pressure from 700 to 1060hPA.

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INSTALLATION

Lift the chair out from the package, without inclining it, and place it on the floor.

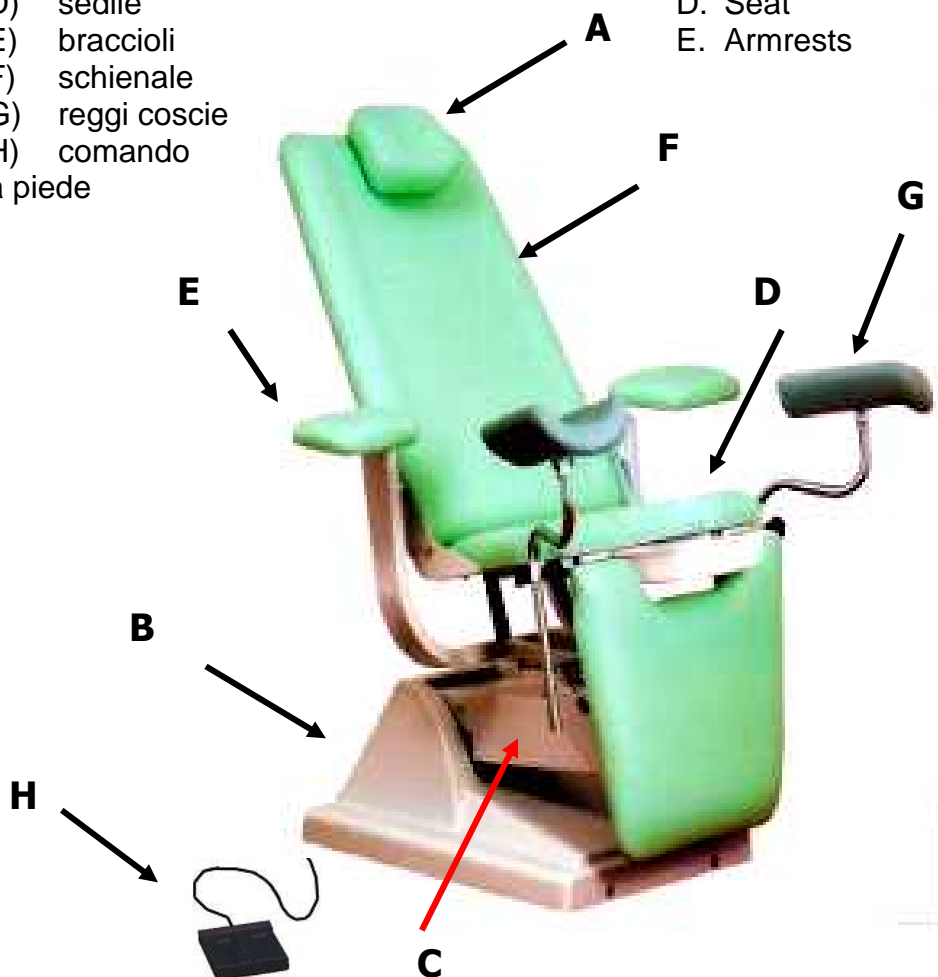
Insert the rubber feet inclining the chair only forward or back, but NEVER LATERALLY (because in that way you could damage basic carter).

Connect the plug power supply.

PARTS DESCRIPTION

The chair is composed by the following parts:

- | | |
|-------------------|-----------------|
| A. Headrest | F. Backrest |
| B. Base | G. Legrests |
| C. Pantograph arm | H. Foot control |
| D. Seat | |
| E. Armrests | |

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WORKING**LIFTING AND LOWERING**

To lift the chair, press the right pedal (B).
To lower the chair, press the left pedal (A).

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BACKREST ADJUSTMENT

Pushing forward A lever, placed under the seat, the adjustment is set free.

To block the backrest on the desired position, release the lever.

If you push the backrest, you will get at the same time seat inclination movement.

Please, kindly note that if the backrest is inclined and the position is set free without the patient on the chair, it is suitable to guide the backrest by a hand in order to avoid a too quick movement given by a gas spring power of thrust without the patient weight.

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LEGREST ADJUSTMENT

When you lift the legrest manually, it remains automatically in the reached position. To lower it, set free the movement by the lever, which is placed under the legrest.

HEADREST ADJUSTMENT

Thanks to the counterbalance, which is inserted inside the backrest, the headrest remains automatically in the same point where you place it.

LEGRESTS ADJUSTMENT

Releasing the knob block, you can adjust high legrest. In the same way, if you rotate support pole, you will get the adjustment of the distance between the right legrest and the left one.

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**MAINTENANCE.
CHAIR CLEANING.**

All materials used to construct the equipments can support cleaning and disinfection operations due in these instructions.

The great variety of drugs chemical products, which are used on medical study can damage painted surfaces and plastic material parts.

Proofs and inquiries made substantiate that surfaces cannot be outright protected by aggression of all products which you can find on the market.

The effects of these products above all depend on time and stay on surfaces.

It is important to remove that chemicals by a damp colth from equipment surfaces immediately.

To clean, please, kindly use a damp colth; do not spray the detergents directly on the equipment.

ATTENTION!

Do not use products which contain alcool, ammonia, abrasive substances, benzene.

The concentrations listed below are the maximum ones that are consented and they cannot be surpassed:

Quaternary Ammonium: maximum 2%

Glutaric Aldehyde: maximum 2%

**EUROCLINIC SRL WILL NOT BE HELD
RESPONSIBLE FOR DAMAGES
CAUSED BY USE OF DISINFECTANTS
WHICH CONTAIN ACTIVE
SUBSTANCES DIFFERENT FROM
THOSE ABOVE MENTIONED.**

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IDENTIFICATION LABEL

Identification label is placed on lifting arm, under the seat.

It is filled in as follows:

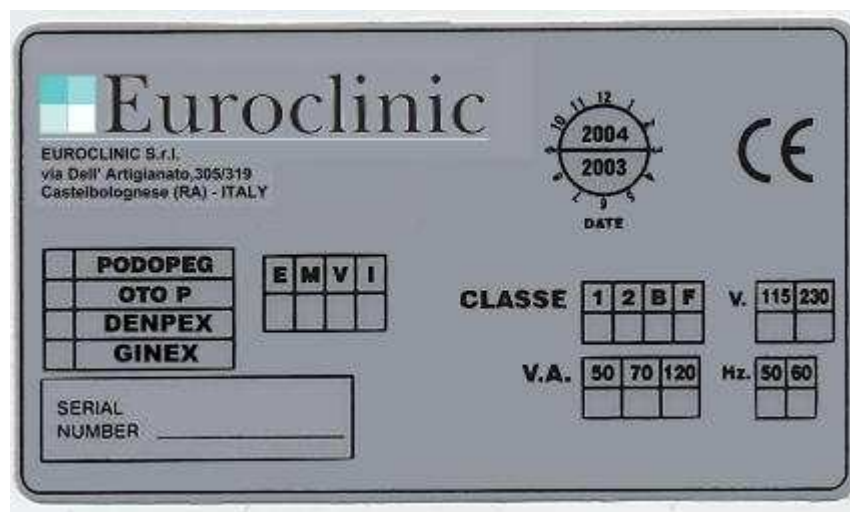
The kind of chair is determined by an incision near the name.

The model is specified by a indelible felt-tippen writing.

On the "clock" above on the right side are engraved construction year and month.

On the table below on the right is mentioned membership class about electrical security, working tension, absorbed power and chair frequency.

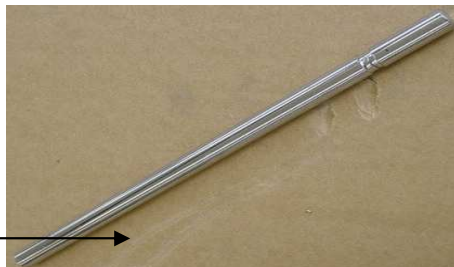
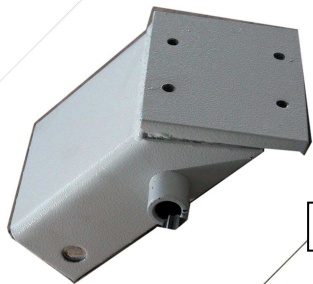
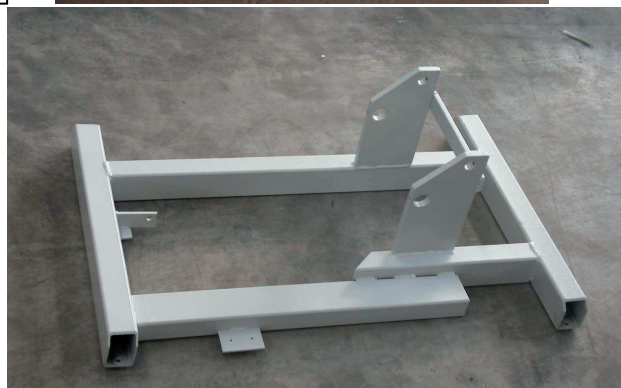
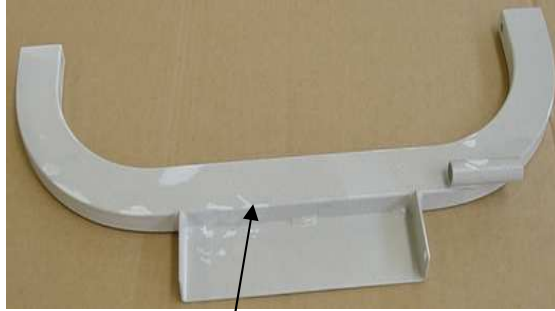
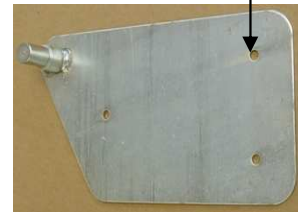
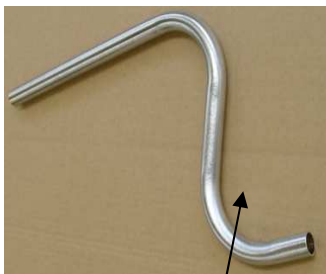
You will find equipment serial number below on the left.

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