GIMA

**OTOSCOPI GIMALUX
GIMALUX OTOSCOPES
OTOSCOPES GIMALUX
OTOSCOPIOS GIMALUX
AURISCÓPIO DE FIBRAS ÓPTICAS
ΩΤΟΣΚΟΠΙΑ GIMALUX**

**MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**

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FEATURES

The otoscope PARKER is medical equipment suitable to directly examine the external acoustic duct and the tympanic membrane.

Unlike the conventional models, this otoscope makes use of a halogen lamp and optic fibres to convey the light beam. It is supplied by two 1,5 V alkaline batteries type AA.

The otoscopes Gimalux are provided with a removable glass allowing a 3X magnification and an output for air insufflation to check the mobility of the tympanic membrane. The otoscopes Gimalux are delivered in an imitation leather case and are equipped with a disposable speculum.

For the otoscopes Gimalux following accessories/spare parts are available:

Code	Accessories for otoscopes Gimalux	Minimum quantity
31478	Halogen lamp for otoscope	-
31491	Disposable mini specula Ø 2,5 mm – black colour	100 pcs..
31492	Disposable mini specula Ø 4 mm – black colour	100 pcs.
31495	Coloured disposable mini specula (various diameter)	1000 pcs.
31496	Disposable mini specula Ø 2,5 mm – blue colour	100 pcs.
31497	Disposable mini specula Ø 4 mm – blue colour	100 pcs.
31498	Disposable mini specula Ø 2,5 mm – green colour	100 pcs.
31499	Disposable mini specula Ø 4 mm – green colour	100 pcs.
31500	Small pump for air insufflation (specify the colour)	-

The product or its components cannot be used for purposes different from the one specified in the present manual.

PRESCRIPTIONS

The product shall be used exclusively with alkaline batteries type “AA” correctly installed. It shall not be dipped into water! When cleaning, make sure that the lamp socket or the battery compartment is not soaked with water.

The product shall be used by qualified medical personnel only, in out-patients' department. It is not conceived for the employment in operation room.

Do not use the equipment in case it is damaged. Apply to your retailer. Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

UNPACKING

Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled. They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

INSTALLATION

Before using it, it is necessary to insert the batteries into the apposite compartment. Proceed as follows:

- Open the battery compartment. To do this counterclockwise turn the apposite ring nut located on the handle.
- Once opened, insert the batteries paying attention to the direction of poles (see picture).
- To close the battery compartment, clockwise screw down the ring nut and check that the batteries are

in contact with poles.

- The operation necessary to remove and replace the speculum is very simple:

Counterclockwise rotate the speculum and draw it out of its seat. To reposition the speculum, insert it into the head group, make the groove match with the small pivot and clockwise rotate.

OPERATION

First of all it is necessary to choose the size and the type of speculum suitable to the acoustic duct to examine.

Besides the ones provided as standard, on request it is possible to deliver disposable specula with different sizes and colours (see table of accessories).

The examiner shall turn on the light by means of the black switch located on the handle upper side.

After the use, make sure that the light is off.

MAINTENANCE

Periodically check the battery conditions, making sure that no sign of corrosion or oxidation is present, in case of necessity replace them with new alkaline ones.

Carefully handle the batteries as the liquids they contain can irritate skin and eyes.

Before being used, thoroughly check the product. The same operation shall be carried out after cleaning.

-Check that the connection between the head and the handle is perfect and that the rheostat correctly works.

-If the light is intermittent or in case it does not turn on, check the bulb, the batteries and the electric contacts.

In case the bulb stops working, it is possible to replace it with original spare parts (cod. 31478).

To replace the bulb, it is necessary to take the speculum out, counter-clockwise turn the bulb and screw the new one down (see picture). It is advisable to use only original bulbs.

Before removing the bulb, make sure that the instrument has been off for some minutes; otherwise you run the risk of being burnt

-Check that the glass is clean; on the contrary it shall be clean with a soft cloth.

-It is advisable to always have new bulbs and batteries at disposal for the maximum light.

-The otoscope head and handle can be cleaned with a cloth soaked with disinfectant, however they cannot be dipped into liquids.

-The speculum used for this otoscope are disposable.

Do not reuse the specula!

Do not sterilize the specula in autoclave or with disinfectants!

After the use, remove the specula from the otoscope head and throw them away!

The product has been conceived to last forever; it is resistant to most part of chemicals except for methyl ethyl ketones (M.E.K.), diluents and nail polish removers. All internal parts are brass and stainless steel plated preventing rusting. If the lamp is exposed to humidity remove the batteries, thoroughly dry to avoid short-circuits.

GUARANTEE

Thank you for having acquired our product. This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty

manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, etc. No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.

The guarantee will be invalidated if the registration number has been removed, cancelled or altered. Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

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